Solo PSV™

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The PSV works by sensing the amount of internal cushion pressure. As air is released, internal pressure decreases and the user is immersed in the cushion for effective pressure distribution.

The PSV is marked with three pre-set positions, which provide the best range of immersion for most users. The therapist determines the PSV position that results in optimal pressure distribution for a client. (A therapist can mark a different position on the PSV if the user falls outside the pre-determined range.)

Sitting on a fully inflated Solo PSV, the client simply opens the PSV to the appropriate position. When the PSV senses that enough air has been released, it automatically shuts off the flow. The client closes the PSV. Adjustment is simple and consistent.

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The Solo uses VARILITE's revolutionary new Pressure Setting Valve (patent pending).

The Solo PSV also is available with a traditional two-way VARILITE valve. Sitting on a fully inflated cushion, the user opens the valve and releases air, closing the valve when sitting on approximately 1/2 inch (13 mm) of air and foam.

5. Cover

The Solo PSV is available with a mesh, non-slip cover for pressure distribution. The underside is a rugged non-slip material with hook and loop for added security. Solo cover meets ISO 7176-16 ignition resistance standards for upholstered wheelchair components. Machine washable.

Ease of use improves user compliance. A therapist can select the Solo PSV with confidence, knowing that a client or caregiver can adjust the Solo PSV easily for optimal pressure distribution. The first time. Every time.

4000 1st Avenue South
Seattle, WA 98134
Phone: (800) 827-4548
Fax: (206) 343-5795
www.varilite.com

Authorized EU Representative:
Cascade Designs, Limited
Dwyer Road, Middleton, Co. Cork
Republic of Ireland

© 2003 Cascade Designs, Inc.